



# WriStable<sup>®</sup> Wrist Brace

Targeted & precise wrist stability providing pain relief without compromising functional range of motion.

## Less is More with the NEW WriStable<sup>®</sup>

The uniquely designed WriStable<sup>®</sup> is the only bracing solution specifically designed to alleviate pain with weight-bearing, scapholunate instability, and dorsal abutment/impingement, while still allowing functional motion!

- Delivers precise wrist support and pain relief while allowing functional range of motion
- Suitable for:
  - Use before and/or after injury
  - Preventing pain during weight-bearing (yoga/gymnastics/push-ups, etc.)
  - Protecting wrist post-injury or due to laxity
  - Providing support for hypermobile wrist when loaded
  - Pain at end range of motion
  - Wrist arthritis
  - Protection during sports & activities
- Features:
  - Revolutionary, sleek design with adjustable wrist stabilization
  - Strap designed for function, comfort, and adjustability
  - Comes in 4 sizes, left and right, to ensure accurate fit
  - Distributes pressure to prevent strangulation
  - Can be worn in water
  - Manufactured in USA from premium durable materials; Machine washable; Latex free




To ensure you experience the full support and pain relief benefits of the WriStable<sup>®</sup>, you must apply the brace correctly & follow the application instructions.

**SIZING: RIGHT or LEFT** Measure around the wrist where the wrist and palm meet

SIZE	INCHES	CENTIMETERS
1 (X-SM)	4 - 5	10 - 12.7
2 (SM)	5 - 6	12.7 - 15.2
3 (MED)	6 - 7	15.2 - 17.8
4 (LG)	7 - 8	17.8 - 20.5



Application Instructions

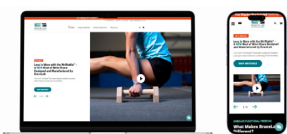

### Ordering with BraceLab

- Order online at [BraceLab.com](https://www.bracelab.com)
- Order through Customer Support via phone at 888-235-8221

**IMPORTANT NOTE:** If you have tingling, numbness, or increased pain while wearing the WriStable<sup>®</sup>, or any redness that does not go away when the brace is off, discontinue wear immediately and seek medical attention.